

Beauty white.



Jet Air Polisher

QUICK JET M



Polishing



Autoclavable



Compact

■ Your patients want white teeth.


The powder jet stream readily removes tobacco tar, tea tannin, stain, and plaque. With Quick Jet M, tooth whiteness and brilliance will be recovered speedily.

Compact design, handpiece type.



Easily portable space-saving design for storage. Attachable to turbine hose and no complicated installation is required. The user-friendly designed grip fits more comfortably in your hand.

Available in quick-disconnect couplings of major brands.

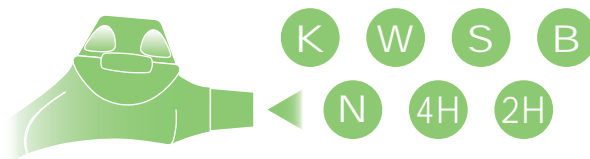

Quick Jet M is available various models in quick-disconnect couplings of major handpiece manufacturers as well as in 2-hole and 4-hole connectors to meet demands of users.



More cleaning efficiency with newly designed head.

The smaller nozzle outlet enables pinpointed and efficient cleaning in a short time. The rotary head easily cleans areas hard-to-reach with conventional jet polishers.



Sterilizable in an autoclave.



All components, not only the head are sterilizable in an autoclave. You can use it safely and hygienically. Inner powder passages can be cleaned from tip to toe easily and quickly.

Available in four colors.



Quick Jet M comes in four colors.

- ivory white
- mint green
- sky blue
- black

It will make your dental office bright and colorful.

Specifications

Model	QUICK JET M - K	QUICK JET M - W	QUICK JET M - S	QUICK JET M - B	QUICK JET M - N	QUICK JET M - 4H	QUICK JET M - 2H
Backend configuration	MULTIflex® coupling	Roto Quick® coupling	Sirona® Quick coupling	Unifix® coupling	Mach®/Phatelus® coupling	Midwest 4-hole	Borden 2/3-hole
Powder consumption volume	4 g/min (Coupling pressure 0.4 MPa)						
Coupling pressure	0.25 to 0.4 MPa (36 to 58 psi, 2.5 to 4.0 bar, 2.5 to 4.1 kgf/cm ²)						
Weight	142g (5.0 oz.)	136g (4.8 oz.)	137g (4.8 oz.)	133g (4.7 oz.)	144g (5.1 oz.)	143g (5.0 oz.)	143g (5.0 oz.)
Length	201mm (7.9 inch)	200mm (7.9 inch)	196mm (7.7 inch)	196mm (7.7 inch)	208mm (8.2 inch)	207mm (8.1 inch)	204mm (8.0 inch)

MULTIflex® is a registered trademark of Kaltenbach & Voigt GmbH & Co. Kg, Germany.
Roto Quick® is a registered trademark of W&H Dentalwerk Burmoos GmbH, Austria.
Sirona® is a registered trademark of Sirona Dental Systems GmbH, Germany.

Unifix® is a registered trademark of Bien-Air Dental S.A., Switzerland.
Mach® and Phatelus® are registered trademarks of NAKANISHI INC., Japan.
Specifications are subject to change without prior notice for improvements.

Manufacturer : **Micron Corporation**

1-34-14 Higashiyukigaya, Ota-ku, Tokyo, 145-0065 Japan
Phone : +81 (3) 3726 0396 Fax : +81 (3) 3726 5396
<http://www.micdent.com/english/>

European Authorized Representative : **EMERGO EUROPE**

Molenstraat 15, 2513 BH, The Hague, The Netherlands
Phone : +31 (0)70 345 8570, Fax : +31 (0)70 346 7299

Contact for inquiries :

Micron Corporation (Overseas Division)

2-17-7 Ikegami, Ota-ku, Tokyo, 146-0082 Japan
Phone : +81 (3) 3755 0396 Fax : +81 (3) 5747 5396
E-mail : office@micdent.com