CASTELLINI S.p.A.
Via Saliceto, 22
40013 CASTEL MAGGIORE
BOLOGNA (ITALY)
Tel +39051700877
Fax +39051701056
E mail: castellini@castellini.com

SILENT POWER

699.1X.A07

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ESSENTIAL PRESCRIPTIONS

The dental patient chair, dental unit (including all the tools and devices connected to it, with the exception of high speed drills), operating lamp, x-ray equipment, compressor and any other device manufactured by Castellini S.p.A. that forms an independent unit must undergo the scheduled annual maintenance operations by technicians carrying a valid Castellini identification card, 365 days after installation, regardless of whether they have actually been used or not.

High speed drills manufactured by Castellini S.p.A. must undergo the scheduled annual maintenance operations performed by qualified technicians authorised by Castellini

The owner of the equipment is responsible for booking the services of the Castellini technician at all times.

The high speed drill must be returned to Castellini S.p.A. three years after installation to be reconditioned at the manufacturing plant.

Subsequently, high speed drills should be reconditioned every three years, in addition to undergoing the above described scheduled annual maintenance operations.

CIRCUIT DIAGRAM, LISTS OF COMPONENTS, CALIBRATION INSTRUCTIONS

Castellini S.p.A. undertakes to provide on request circuit diagrams, lists of components, calibration instructions or any other information that may be required by qualified technicians, authorized by Castellini and in possession of a valid professional licence, to repair those parts of the equipment that may be repaired.

Castellini S.p.A. reserves the right to modify the products at any moment without notice.

ATTENTION

- During the working, this unit will not cause radio interference to the electric network.

SCRAPPING

If scrapped, the equipment must be disposed of in accordance with the provisions of applicable legislation.

The materials used to construct the equipment do not constitute a hazard for humans or animals in the event of contact or exposure.

WARNING

The **Silent Power** high speed air driven turbines are class IIa medical devices intended to be fitted on dental equipments, built in compliance with 93/42/EEC Directive and with IEC 601-1, EN 60601-1 regulations.

Install such instruments only on units in conformity with the above said standard!

It is indispensable to read the instructions of this manual carefully before using these handpieces.

The high speed air driven turbine is intended to be used by Dentists only, who may avail themselves of the assistance of authorized personnel.

It must be operated in environmental conditions in accordance with the prescriptions of the dental unit user's manual.

The Manufacturer will in no case be held responsible for the safety, reliability and performance of the device where:

- the premises do not satisfy the basic requirements (as described in the dental unit user's manual);
- the assembly, the fitting of additional devices, the calibration and any successive adjustments or repairs are not carried out by qualified technicians authorized by Castellini S.p.A. and in possession of a valid professional license;
- any unauthorized modifications, tampering or incorrect servicing are carried out or where non-original spare parts and/or components are used;
- the device is not used in accordance with the operating instructions or is used for purposes other than its intended use.

TECHNICAL SPECIFICATIONS

	Silent Power Gold Silent Power Silver 4L	Silent Power Gold Miniature	Silent Power Silver 2 Silent Power Silver 2L
Speed	350.000 ÷ 366.000 rpm'	380.000 ÷ 400.000 rpm	350.000 ÷ 366.000 rpm'
Air pressure (handpiece)	260 ÷ 280 kPa (2.6 ÷ 2.8 bar)	280 ÷ 300 kPa (2.8 ÷ 3 bar)	260 ÷ 270 kPa (2.6 ÷ 2.7 bar)
Air delivery	48 ÷ 51 lt/min	44 ÷ 47 lt/min	48 ÷ 51 lt/min
Water pressure	70 ÷ 140 kPa (0.7 ÷ 1.4 bar)	70 ÷ 140 kPa (0.7 ÷ 1.4 bar)	70 ÷ 140 kPa (0.7 ÷ 1.4 bar)
Burr stem diameter	1.590 ÷ 1.600 mm (ISO 1797-1)	1.590 ÷ 1.600 mm (ISO 1797-1)	1.590 ÷ 1.600 mm (ISO 1797-1)
Max burr length	26 mm	19 mm	26 mm
Min burr lock length	11.7	11.2	11.7
Max burr diameter	2 mm	2 mm	2 mm

Warning!

- **High speed handpiece!**
- **An excessive pressure may damage the tooth!**
- **Cool the operating area conveniently! Stop the turbine immediately if the cooling media don't flow!**
- **Do not use burs with dimensions exceeding the above mentioned limits! Do not use worn or damaged burs! Handle burs with care wearing protective gloves!**
- **Only CE-marked burs (or other similar tools) according to Directive 93/42/EEC must be used.**
- **Before every use, check that the burr is correctly locked in the gripper (see section "BURR LOCKING").**
- **The burr release button becomes very hot if it is pushed while the turbine is running. Pay the greatest attention not to push it unintentionally against the Patient's oral cavity.**
- **Do not use the instrument if it is visibly damaged or if unusual noises and/or vibrations are produced. Contact an authorised technical service centre.**
- **For the use of this instrument the rubber dam must be applied on the patient.**

The high speed air turbine has to be subjected to intermittent operation: 20 min work, 10 min stand-by.

HANDPIECE CONNECTION

a) Silent Power Gold, Silent Power Gold Miniature, Silent Power Silver 4L (Pict.1)

The handpiece must be connected to its hose by means of Castellini faston coupling (1 – pict. 1) which has four ways plus two electric contacts coupling section.

Screw in the faston tightly to the flexible handpiece tubing. Lubricate O-Rings with specific S1 lubricant. Fit the handpiece over the faston coupling and push until it clicks.

To remove the handpiece, pull it back away from the tubing. Don't remove the handpiece while the burr is rotating!

b) Silent Power Silver 2, Silent Power Silver 2L

Fit the handpiece to the tubing and tighten the ring.

REGULATING DRIVING AIR PRESSURE (Pict. 2)

Connect the fitting with the pressure gauge (2 – pict. 2) between the tubing and faston or between the tubing and the handpiece, for handpieces with screw connection. Operate the high-speed handpiece and check pressure (Pict. 2).

Driving air-pressure: 270 ÷ 280 kPa (2,7 ÷ 2,8 bar) for Silent Power Gold and Silver 4L.

280 ÷ 290 kPa (2,8 ÷ 2,9 bar) for Silent Power Gold Miniature.

260 ÷ 270 kPa (2,6 ÷ 2,7 bar) for Silent Power Silver 2 and 2L.

REGULATING SPRAY WATER (Pict. 3)

Use the ring on the tubing coupling to control water supply.

BURR LOCKING (Pict. 4)

Simply press down with thumb in the centre of handpiece head back cover to remove and introduce a new burr, and lift thumb to lock in place.

Check the correct fitting of the burr pulling it by hand. Wear protective gloves!

Push burr all the way in for correct fitting. Always use burs with 1.590-1.600 mm. standard stem (ISO 1797-1).

To prevent damage, NEVER release burr until high-speed handpiece stops turning.

REPLACING HIGH SPEED HANDPIECE ROTOR (Picts. 5 - 6)

The rotor is field replaceable. Unscrew cap of head (4) with the key (pict.5). Before fitting a new rotor, make sure that O-Rings (5) are in their correct position, i.e. in the head and cap.

Now screw on cap tightly.

When replacing the rotor replace also the O-Rings and the wavy washer in the cap (Pict. 6).

LIGHTING DEVICE

The halogen microlamp on the faston coupling has to be powered with a **maximum 3.5 Vdc** source. NEVER touch it with bare hands. If this occurs, clean lamp with cotton wool and alcohol.

Use the control on the dental unit table to adjust light intensity. The lamp lights automatically when the high-speed handpiece is working: when it stops, the light will stay on for a set time (up to 20 sec. maximum).

"SILENT POWER"**REPLACING LAMP****UK****a) Silent Power Gold, Silent Power Gold Miniature, Silent Power Silver 4L (Pict. 7)**

Disconnect handpiece from faston coupling. Unscrew the front part (9) of the faston coupling and remove bulb. Put a new bulb in place and screw on part n. 9 again.

b) Silent Power Silver 2, Silent Power Silver 2L (Pict. 8)

Disconnect handpiece from tubing, slide off seal (10) and remove bulb (11). Put a new bulb in, put seal (10) in position (make sure holes are properly positioned) and connect handpiece to tubing again.

LUBRICATING HIGH SPEED HANDPIECE

Use the spray-can "DAILY OIL" to **lubricate at least once daily and in the hereafter indicated circumstances the high-speed handpiece**. Follow the instructions on can, remembering to apply lubricant in two short sprays. Eliminate any excess lubricant before use by operating the handpiece (hold it over cuspidor) and wiping in with gauze and cotton. Do this with burr in place to prevent mechanical damage to handpiece.

In addition it is imperative to lubricate as follows:

-Before and after autoclaving

-Before and after every AUTOSTERIL disinfecting or sterilizing cycle.

Regular use of this spray lubricant assures best working order.

Proceed as shown in Picts. 9 and 10, respectively for faston coupling and for screw coupling.

CLEANING, DISINFECTING, STERILIZING

Wear protective gloves! Remove the burr!

Use gauze or cotton soaked in surgical alcohol to clean and/or disinfect handpiece exterior. **DO NOT soak handpiece directly in solution.** DO NOT use an ultrasonic cleaner.

Clean the spray nozzle after each operation: use the steel wire supplied and the syringe to blow on the nozzle. Clean fibre-optic terminals with alcohol-soaked gauze.

In order to maintain a proper level of hygienic safety handpieces must be sterilized in autoclave with water steam or chemical steam up to 135°C, 210 kPa (2.1 bar) for 20 min, after the use on each patient.. Never sterilize handpieces in dry-heat sterilizers.

Before and after each autoclaving, lubricate handpiece and clean fibre-optic terminals. Never leave handpieces in autoclave overnight: always remove them from autoclave after each cycle.

The instrument is supplied not sterile. Before use, sterilize according to the above specifications.

Warning! Check the autoclave periodically according to the manufacturer's prescriptions! Temperature exceeding the above stated limit may damage the handpiece!

SERVICING THE CASTELLINI FASTON CONNECTION

Lubricate at least weekly the O-Rings with silicone lubricant S1, distributed by Castellini S.p.A. Wearing protective disposable gloves, apply a small amount of S1 on fingers and use it for lubricating.

If faston coupling leaks, replace the O-Rings: grasp them with two fingers and slide them along one side and forward to remove them. Slip on the new rings, fit them properly into the grooves and lubricate.

Important! The faston coupling **CANNOT** be sterilized in autoclave.

IMPORTANT

***WE PRESCRIBE TO
STOP WORKING IMMEDIATELY IN THE EVENT OF
UNUSUAL NOISE, STRONG VIBRATION OR IF BURR IS
NOT KEPT FIRMLY IN POSITION.***

TROUBLESHOOTING

PROBLEM	LIKELY CAUSE	REMEDY
Low power; abrupt rpm loss	Irregular air pressure	Check and adjust
	Air leaks at tubing coupling	Tighten coupling ring or replace O-Rings if worn
	Blocked air way	Check tubing or replace it
	Handpiece needs lubrication	Lubricate repeatedly between brief runs
Very noisy handpiece	Worn out bearings	Replace rotor assembly
	Misaligned or very worn burr	Fit in new burr
Defective attaching or detaching burr:	Worn out bearings	Replace rotor assembly
	Not-calibrated burr diameter	Use only quality burrs
	Curved burr	Use new burr
	Worn clamp	Replace rotor assembly
Defective spray	Button not fully pressed down	Push the button at centre all the way
	Blocked nozzles	Detach handpiece, hold tube over cuspidor and blow out air and water. N.B. Water regulator must be open Clean handpiece: use supplied steel wire carefully so as not to ovalize nozzles.
No light in fibre-optics handpiece	Burned out lamp	Replace lamp
	Defective tube coupling	Tighten fully the handpiece tube coupling ring, or, if not enough, replace tube assembly
Water leaks from coupling	Light control at minimum	Adjust control on the unit table
	Loose tubing	Tighten fully coupling ring
	Worn faston O-Rings	Replace and lubricate them.

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In respect of:

For the design, development, manufacture and installation of dental equipment, x-ray generators and their supports and stands, electrosurgical units, and dental handpieces

On the basis of our examination under the requirements of Council Directive 93/42/EEC, Annex II, Section 3.2.

For and on behalf of the British Standards Institution, a Notified Body for the above Directive (Notified Body Number 0086):

Anne Boyd, Divisional Director, Product Services Operations

Conditions of Approval

Validity of this certificate is conditional on the quality system being maintained to the requirements of the Directive. This approval excludes all products designed and/or manufactured by a third party on behalf of the company named on this certificate unless specifically agreed with BSI.

This approval excludes Class 3 medical devices.

First Issued: **11 December 1995** Date: **26 January 2004**Expiration Date: **10 December 2005****BSI Product Services**Maylands Avenue, Hemel Hempstead, Hertfordshire HP2 4SQ United Kingdom
Tel: +44 (0)1442 230442 Website: www.bsi-global.com
BSI Group Headquarters: 389 Chiswick High Road, London W4 4AL Tel: +44 (0)208 996 9000

Page 1 of 1

**CERTIFICATE OF REGISTRATION****Quality Management System**

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Hold Certificate No: **FM 51210**

and operate a Quality Management System which complies with the requirements of BS EN ISO 9001:2000 for the following scope:

The design, manufacture, installation, maintenance and service of "own brand" dental equipment, X-ray units, electrosurgical units and dental handpieces. Provision of training and qualification for internal and external service engineers to install, maintain and service company products.

For and on behalf of BSI:

Certification Manager, Systems Assessment

Originally registered: **9 Jul 1999**

Latest issue: **30 Dec 2003**

Page: 1 of 1

003

This certificate remains the property of BSI and shall be returned immediately upon request. This certificate does not expire. To check its validity telephone: +44 (0)20 8996 9001 or visit www.bsi-global.com/ClientDirectory. Further clarifications regarding the scope of this certificate and the applicability of ISO 9001:2000 requirements may be obtained by consulting the organization.

*The British Standards Institution is incorporated by Royal Charter.
Group Headquarters: 389 Chiswick High Road, London W4 4AL, UK.*

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*Operate a Quality Management System
which complies with the requirements of
BS EN ISO 9001:2000
for the activities detailed in the scope of
registration.*

Certificate No: **FM 51210**

Signed on behalf of BSI

Originally registered: **9 Jul 1999**

003

*This is not a legal document and cannot be used as such. This certificate does not expire.
To check its validity telephone +44 (0)20 8996 9001 or visit www.bsi-global.com/ClientDirectory
Further clarifications regarding the scope of this certificate and the applicability of ISO 9001:2000 requirements may be
obtained by consulting the organization. The British Standards Institution is incorporated by Royal Charter.
Group Headquarters: 389 Chiswick High Road, London W4 4AL, UK.*

MC4402/ISSUE1/SA/0102/UK/DP

**CERTIFICATE OF REGISTRATION****Quality Management System**

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Hold Certificate No: **MD 76089**

and operate a Quality Management System which complies with the requirements of **ISO 13485:2003** for the following scope:

The design, manufacture, installation, maintenance and service of "own brand" dental equipment, x-ray units, electrosurgical units and dental handpieces. Provision of training and qualification for internal and external service engineers to install, maintain and service company products to maintain compliance of products

For and on behalf of BSI:

Alastair Trivett, Managing Director, BSI Product Services – Global

First Issued: 23 Jan 2004

Date: 30 Oct 2006

Expiration Date: 22 Jan 2010

Page 1 of 1

This certificate remains the property of BSI and shall be returned immediately upon request.

The British Standards Institution is incorporated by Royal Charter.

Group Headquarters: 389 Chiswick High Road, London W4 4AL, UK.

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UK

*For technical assistance activities,
spare parts /reparations
it's necessary a apply to the authorized
Technical Assistance Centers
or directly to
CASTELLINI S.p.A.
Via Saliceto, 22
40013 CASTELMAGGIORE
BOLOGNA (ITALY)
Tel (int. code) + 39(0)51700877
Fax (int. code) + 39(0)51701056
E mail: castellini@castellini.com*

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